REF: AD11M CINTURÓN ABDOMINAL CIERRE MAGNÉTICO

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EN REF: AD11M ABDOMINAL FIXING BELT FOR CHAIR WITH MAGNETIC CLOSURE

Class I Medical Device, according to the European directive 2017/745 on medical devices and as such complies with the essential requirements that apply to it, and must be used under the control of a medical specialist or orthopaedic technician to achieve effectiveness in its therapeutic purpose and safety in its use, i.e. that its use does not represent any risk to the user's health.

To ensure the correct functioning and optimal performance of this medical device, read the instructions carefully before using it for the first time. Check the correct external appearance of the product and its integrity, as well as the integrity of the packaging.

Package contents: 1 chair belt, 1 lock: MAD55, 1 key: LAD55, 1 instruction manual, 1 advertising sheet.

INDICATIONS: Comfortable fastening of the patient to the wheelchair, preventing possible falls and slipping. Possibility of extension for personalised adaptation in combination with the CAD16M extension.

CONTRAINDICATIONS: The product is contraindicated in case of patient agitation and for patients who may become detached from the device. Do not use in case of allergy to any of the components.

SAFETY INFORMATION: Do not use the ADIFIX positioning and restraint system without having read and understood the instructions. Any misuse of the system may result in the risk of serious injury or death. The safety of your patients depends on the correct installation and use of the ADIFIX system. The ADIFIX system must always be used by qualified healthcare personnel in accordance with the manufacturer's instructions and in compliance with the applicable state and EU regulations. Any private user must strictly comply with the doctor's orders. The condition of the product must be checked regularly. No modifications must be made to the original product. This product must not come into direct contact with an injured area. Frequent checking and verification of the correct fit of the product on the user is recommended. Never leave the device within the reach of children and/or persons who cannot look after themselves. Due to the textile nature of the components, do not expose the product to a heat source. Liquids cause failure of the fasteners; do not immerse the fasteners in any kind of liquid. In case the locking mechanism cannot be opened, the strips can be cut with scissors. Warning for pacemaker patients! A distance of at least 10 cm must always be maintained between the patient's pacemaker and the ADIFIX magnetic key, otherwise the patient may experience tachycardia. No side effects are known.

INSTALLATION INSTRUCTIONS:

- 1º Seat the user in the chair.
- 2º Place the padded part on the user's abdomen, taking the straps at the back of the chair.
- 3º Place the magnetic button in the eyelets joining the two straps by joining the two strips at the desired size.

OPENING AND CLOSING ADIFIX:

Open the ADIFIX lock (MAD 55):

4. Place the magnetic key (LAD55) on the button on the top of the lock (MAD55).
5. Make sure that when opening the ADIFIX lock, the lock is not under pressure from the tapes - it may not open.
6. Lift the magnetic key together with the black button to open the lock.

Close the ADIFIX lock (MAD55):

1. Place the black button on the piston.
2. Check that the lock is locked. You should not be able to open the lock.

SIZE:

REF	SIZE	WIDTH	LENGTH
AD11M	0	12,5 cm	40 cm
AD11M	1	16,5 cm	56 cm

COMPOSITION: 100% polyester laminated fabric. Retirer tape 100% polyester. 100% nylon thread. Eyelet and Washer: nickel plated brass.

STORAGE AND CARE: Machine wash at 60° maximum. Do not dry clean or iron. Do not tumble dry. Do not bleach. The belts should be washed in washing machines that do not exert pressure on the metal parts of the system, otherwise these parts may deteriorate. Before each washing and drying, remove all ADIFIX fasteners in order to avoid damaging them and causing their failure. Dirty fasteners can be cleaned with a damp cloth. The seals must not be immersed in liquids of any kind.

Store the product in a dry place, protected from light, at room temperature in its original packaging.

REUSE: Given its therapeutic purposes, this product is not reusable by another person.

WASTE MANAGEMENT: As far as waste management is concerned, the product and its components comply with the provisions in force laid down by the local authorities.

WARRANTY: 24 months from the date of purchase. You must keep the invoice as proof of purchase for any claim. The product is intended exclusively for the use described in this manual. Any damage resulting from improper use is excluded from the warranty. Damages are limited to the value of the goods. Compensation for consequential damages is expressly excluded.

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient are established.

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