REF: AD15 CHALECO SUJECCIÓN SILLA

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REF: AD15 FIXING VEST FOR CHAIR

Class I Medical Device according to the European Medical Device Directive 2017/745 and as such complies with the essential requirements that apply to it.

To ensure the correct functioning and optimal performance of this medical device, please read the instructions carefully before using it for the first time. Check the correct external appearance of the product and its integrity, as well as the integrity of the packaging. Package contents: 1 chair waistcoat, 1 instruction manual, 1 advertisement sheet.

INDICATIONS: Provides adequate trunk stability for users with low muscle tone. Prevents lateral displacement.

CONTRAINDICATIONS: The product is contraindicated in case of patient agitation and for patients who can detach from the device or can unlock the buckles on their own. Do not use in case of allergy to any of the components.

SAFETY INFORMATION: It is recommended that the product be installed by qualified and trained personnel. The condition of the product should be checked regularly. No modifications should be made to the original product. This product should not be placed in direct contact with an injured area. Frequent checking and verification that the product is being used correctly by the user is recommended. Never leave the device within the reach of children and/or persons who cannot take care of themselves. Due to the textile nature of the components, do not expose the product to a source of heat or moisture. There are no known side effects.

INSTALLATION INSTRUCTIONS:

- 1º sit the user in the chair.
- 2º Place the waistcoat on the patient.
- 3º Join the lower straps by closing the buckle and adjust the necessary size.
- 4º The strap at the back of the vest should be put through the strap at the front and close the buckle at the back. See side photo.


SIZE:

SIZE	DIMENSION
0	29X40 CM
1	39X59 CM

COMPOSITION: 100% polyamide mesh fabric. Retirer tape 100% polyester. Buckle: Acetal. Thread: 100% Polyester.

STORAGE AND STORAGE: Machine wash at 60° maximum. Do not dry clean or iron. Do not tumble dry. Close buckle when washing. Do not bleach.

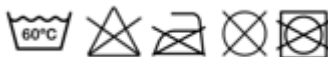Store in a dry place, protected from light, at room temperature in its original packaging.

REUSE: Due to its therapeutic purposes, this product is not reusable by another person.

WASTE MANAGEMENT: As far as waste management is concerned, the product and its components comply with the provisions in force laid down by the local authorities.

WARRANTY: 24 months from the date of purchase. You must keep the invoice as proof of purchase for any claim. The product is intended exclusively for the use described in this manual. Any damage resulting from improper use is excluded from the warranty. Damages are limited to the value of the goods. Compensation for consequential damages is expressly excluded.

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient are established.

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